# Aquatec® Ocean VIP / VIP XL / Dual VIP

en	<b>Shower and Toilet Commode</b> User Manual .....	3
de	<b>Dusch- und Toilettensitz</b> Gebrauchsanweisung.....	27
fr	<b>Chaise roulante pour douche/toilettes</b> Manuel d'utilisation .....	51
nl	<b>Douche- en toiletrolstoel</b> Gebruiksaanwijzing .....	75
it	<b>Carrozzina da doccia e da toilette</b> Manuale d'uso.....	99
es	<b>Silla de ducha e inodoro</b> Manual del usuario.....	123
pt	<b>Cadeira sanitária de chuveiro e sanita</b> Manual de utilização .....	147

This manual **MUST** be given to the user of the product.  
**BEFORE** using this product, read this manual and save for future reference.

**AQUATEC®**

**Yes, you can.®**

© 2015 Invacare® Corporation

All rights reserved. Republication, duplication or modification in whole or in part is prohibited without prior written permission from Invacare. Trademarks are identified by ™ and ®. All trademarks are owned by or licensed to Invacare Corporation or its subsidiaries unless otherwise noted.

---

# Contents

---

This manual **MUST** be given to the user of the product.  
BEFORE using this product, read this manual and save for future reference.

<b>1 General</b>	<b>4</b>
1.1 General information	4
1.2 Symbols in this user manual	4
1.3 Warranty	4
1.4 Standards and regulations	4
1.5 Intended use	5
1.6 Service life	5
<b>2 Safety</b>	<b>6</b>
2.1 Safety information	6
2.2 Labels and symbols on the product	7
<b>3 Components</b>	<b>9</b>
3.1 Overview	9
<b>4 Setup</b>	<b>10</b>
4.1 Safety information	10
4.2 Assembling the shower and toilet commode	10
<b>5 Usage</b>	<b>15</b>
5.1 Safety information	15
5.2 Using the shower and toilet commode	16
<b>6 Transport</b>	<b>19</b>
6.1 Safety information	19
<b>7 Maintenance</b>	<b>20</b>
7.1 Safety information	20
7.2 Cleaning and disinfection	20
7.3 Changing the castors	21

<b>8 After Use</b>	<b>22</b>
8.1 Storage	22
8.2 Re-use	22
8.3 Disposal	22
<b>9 Troubleshooting</b>	<b>23</b>
9.1 Identifying and repairing faults	23
<b>10 Technical data</b>	<b>24</b>
10.1 Dimensions and weight	24
10.2 Materials	25

## I General

### I.1 General information

This user manual contains information and notes on the proper and safe use of the shower and toilet commode.

Please read this user manual carefully before using the product. Please observe all the notes, particularly the safety information, and act accordingly.

The position specifications (A, B, C, etc.) in the handling instructions always refer to the preceding figure.

### I.2 Symbols in this user manual

In this user manual, warnings are marked with symbols. The title next to the warning symbol indicates the level of danger.

#### WARNING

Indicates a dangerous situation which could lead to death or serious injury if not avoided.

#### CAUTION

Indicates a dangerous situation which could lead to minor injuries if not avoided.

#### IMPORTANT

Indicates a dangerous situation which could lead to damage if not avoided.

Useful tips, recommendations and information to ensure efficient and smooth operation.

This product complies with the directive 93/42/EEC for medical products. The launch date for this product is specified in the CE declaration of conformity.

Manufacturer

### I.3 Warranty

We provide a manufacturer's warranty for the product in accordance with our General Terms and Conditions of Business in the respective countries. Warranty claims can only be made through the provider from whom the appliance was obtained.

### I.4 Standards and regulations

Quality is of the utmost importance for our company; all processes are based on the ISO 9001 and ISO 13485 standards.

This product features the CE mark, in compliance with the Medical Device Directive 93/42/EEC Class I.

The product meets the requirements of the standards DIN EN 12182 (Technical aids for disabled persons - general requirements and test methods).

Invacare® is continuously working towards ensuring that the company's impact on the environment, locally and globally, is reduced to a minimum. We only use REACH compliant materials and components.

For further information please contact Invacare® in your country (for addresses, see back page of this manual).

## **I.5 Intended use**

The shower and toilet commode is intended solely as an aid when showering, using the toilet or transferring patients in an indoor environment. Any other use is prohibited.

You can use the product as a toilet commode by placing it over a standard toilet or by combining it with a commode pan (optional).

Operating temperature: 5 - 40 °C.

## **I.6 Service life**

We estimate a life expectancy of five years for these products, provided they are used in strict accordance with the intended use as set out in this document and all maintenance and service requirements are met. The estimated life expectancy can be exceeded if the product is carefully used and properly maintained, and provided technical and scientific advances do not result in technical limitations. The life expectancy can also be considerably reduced by extreme or incorrect usage. The fact that we estimate a life expectancy for these products does not constitute an additional warranty.

## 2 Safety

### 2.1 Safety information

#### **WARNING!**

- Do not use this product or any available optional equipment without first completely reading and understanding these instructions and any additional instructional material such as user manuals, service manuals or instruction sheets supplied with this product or optional equipment. Invacare product manuals are available on the internet or your local dealer (Addresses are displayed on the back cover of this manual).
- If you are unable to understand the warnings, cautions or instructions, please contact a healthcare professional, dealer or technical personnel before attempting to use this equipment – otherwise, injury or damage may occur.

#### **WARNING!**

##### **Risk of injury**

- Do not stand on the product.
- Do not use as a climbing aid.
- Do not use outdoors.

#### **WARNING!**

##### **Risk of injury**

- Do not use if defective.
- In the event of a malfunction, contact your dealer immediately.
- Do not make any unauthorized changes or alterations to the product.
- Only use accessories intended for this product.

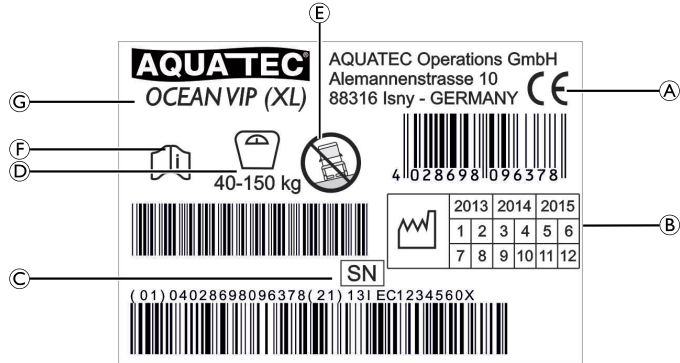
#### **IMPORTANT!**

- The information contained in this document is subject to change without notice.
- Check all parts for shipping damage and test before using.

## 2.2 Labels and symbols on the product

### Identification label

The identification label contains important information:

(A)	Conformity mark
(B)	Product time frame
(C)	Serial number
(D)	Load capacity
(E)	Note (use only on flat surfaces)
(F)	Note (observe accompanying documents)
(G)	Device designation



The identification label is attached to the inside of the right side section.

### Labels on the product

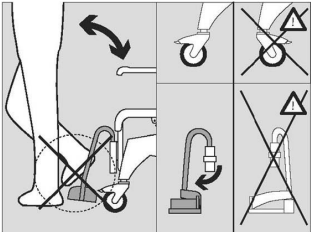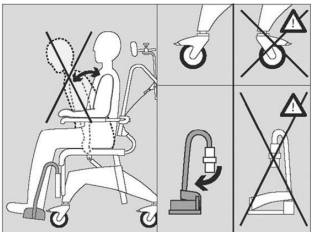Following labels are attached to the backside of the headrest:

Label	Description
	<p>Correct usage when swivelling the seat frame with backrest, ® 5.2 Using the shower and toilet commode, page 16.</p>
	<p><b>CAUTION!</b></p> <p>Risk of hitting</p> <p>Your body could be hit by the backrest when pulling the removable handle.</p> <p>– Make sure there is enough space between you and the backrest when swivelling the seat frame with backrest.</p>

Following labels are attached to the side parts:


Label	Description
	<p><b>WARNING!</b></p> <p>Risk of tipping over</p> <ul style="list-style-type: none"> <li>– Do not step on the footrest when getting into or out of the shower and toilet commode.</li> <li>– Push the footrests to the sides or fold up the foot plates before getting into or out of the shower and toilet commode.</li> <li>– Before sitting down or standing up, turn the casters so that the lock is pointing forwards.</li> </ul>
	<p><b>WARNING!</b></p> <p>Risk of tipping over</p> <ul style="list-style-type: none"> <li>– Do not lean too far forwards when sitting in the shower and toilet commode.</li> <li>– Attach front antitippers.</li> </ul>

## 3 Components
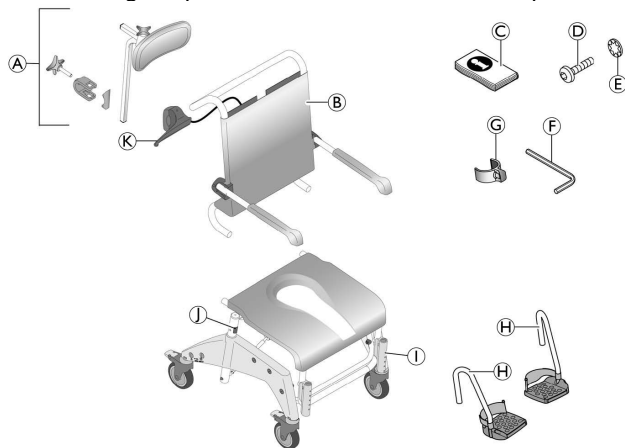
### 3.1 Overview

The following components are included within the scope of delivery:

ⓐ	Seat frame with seat plate and side sections with casters
ⓑ	Clips (2 x) for height adjustment (inserted from the inside)
ⓒ	Removable handle

ⓐ	Head support
ⓑ	Backrest with cover and armrests
ⓒ	User Manual (in plastic bag)
ⓓ	Screws (4 x) for backrest
ⓔ	Tooth lock washer (4 x) for backrest
ⓕ	Allen key
ⓖ	Clips (2 x) for footrests (in plastic bag)
ⓗ	Footrests with heel straps (2 x)

## 4 Setup

### 4.1 Safety information

#### WARNING!

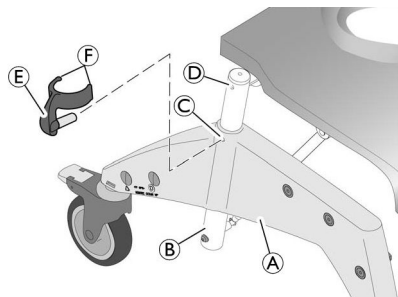
- Check the parts for transport damage before commissioning and contact your dealer if required.
- Ensure during assembly that the parts are positioned correctly with respect to one another.

This section explains how to assemble the product.

The product is dismantled in the reverse order.

### 4.2 Assembling the shower and toilet commode

#### Fitting the seat frame tube for height adjustment

- I. Pull the two clips (E) for height adjustment from the inside (D) of the seat frame (A) and insert into the side section (C) from the outside until both lugs (F) enclose the seat frame tube (B).

### Adjusting the seat height

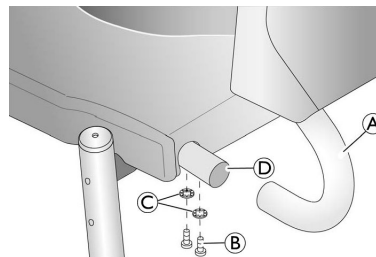
#### WARNING!

- The seat height must be adjusted to suit the user. As a rule, the seat height is correctly adjusted when the user is sitting on the commode and both feet touch the ground.
- The seat height must be adjusted on both sides simultaneously. To make adjustments, a second person is required to help.
- Do not adjust the seat height of the commode when someone is sitting on it.

1. Slightly raise the commode at the seat frame.
2. Take out both clips (E).
3. Pull the seat frame out of the side section (A) or push it in.
4. Align the clip recesses (C) at the side section with a hole on the seat frame (D).
5. Push both clips from the outside into the side section until the two lugs (F) enclose the seat frame tube (B).

### Fitting the backrest

1. Push down all four red foot pedals for the casters to lock them.

**!** **IMPORTANT!**

- Make sure the armrests are facing forward when folded up.

2. Push the backrest (A) from behind onto the pin on the seat frame (D) until the collar of the backrest touches the seat frame tube.

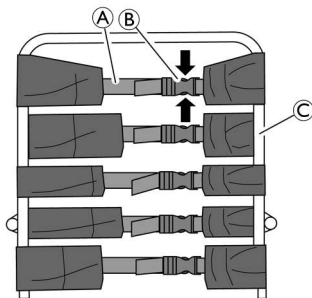
**!** **IMPORTANT!**

- Tighten the screws using the Allen key supplied. The torque is 5 Nm.
- Regularly check that the screws are firmly fitted.

3. Tighten the backrest on the left and right sides using two screws (B) on each side and tooth lock washers (C).

### Adjusting the backrest cover

The tension of the backrest can be adjusted as desired.

**WARNING!**

- Make sure that at least three belts are always passed by the backrest frame (on the outside) and their belt locks are closed.

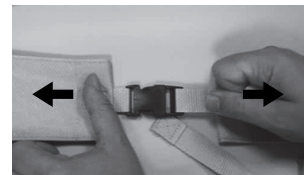
To attach accessories, the second belt lock can be opened from above and the backrest cover can be passed by the backrest frame on the inside at point C.

Tightening:

1. Pull the belts (A) of the backrest cover tighter.

Loosening:

1. Open the buckles (B) by squeezing the retaining lugs together (arrows).
2. Push the belts back a little through the buckle.
3. Close the belt buckles.
4. Retighten the belts gently.


**WARNING!**

**Risk of injury due to damaged backrest buckles!**

Check for secure fastening of the backrest buckles each time the product is maintained or reconditioned.

- Check all buckle clips for secure fastening by closing the buckles and pulling on both ends.
- Replace the backrest/backrest buckles if a buckle opens under the applied force.

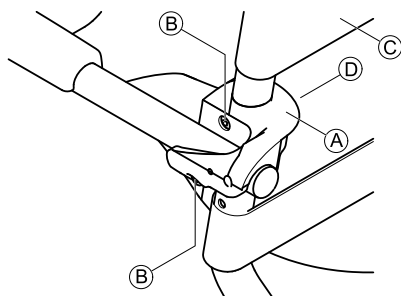**Adjusting the height of the armrests**

**IMPORTANT!**

- Adjust the height of the armrests to suit the user.
- Change the height of the armrests evenly on both sides.
- Set the height of the right and left armrest in the exact same manner.

1. On the backrest cover, open the buckle of the belt that passes on the inside of the backrest frame.
2. On the backrest cover, open the buckle of the belt © that runs above the armrest.
3. Pass this belt on the inside of the backrest frame and re-close.

The nuts are inserted loosely in the retaining clamp and can fall out.

4. Unscrew the screws (B) of the armrest bracket (A) using the Allen key supplied.
5. Remove the securing clamp (D) with the nuts from the inside of the backrest frame.
6. Remove the backrest bracket (A) with the armrest and move it upwards.

The smaller hole serves to fix the retaining clamp.

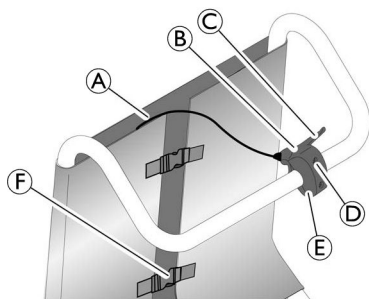
7. Insert the screws in the armrest bracket and guide them through the holes in the backrest frame.
8. If required, put the nuts back into the retaining clamp.
9. Place the retaining clamp on the backrest frame so that the fixing on the retaining clamp projects into the smaller hole.
10. Turn the screws in the nuts and tighten.
11. Pass the belt of the backrest cover that is still open on the outside of the backrest frame, close and tighten all belts.

**IMPORTANT!**


- The armrests are lowered using the same procedure.
  - When converting to the lower position, the belt that lies beneath the armrest must be opened and passed inside the backrest frame.

## Fitting the removable handle

1. Unscrew the two cross slotted cheese head screws **D** from the removable handle **B** and remove the lower part of the fastening clamp **E**.
2. Guide the removable handle with the wire **A** for activating the pneumatic springs upwards through the gap between the backrest cover and the belt locks **F**.
3. Place the removable handle with the top part of the fastening clamp onto the handle of the backrest so that the hand lever **C** points to the right.
4. Fix the removable handle to the tubular frame using the bottom part of the fastening clamp and the two cross slotted cheese head screws. Tighten the screws lightly.
5. Align the removable handle as shown in the figure above.
6. Tighten the two cross slotted cheese head screws.

## Fitting the footrests

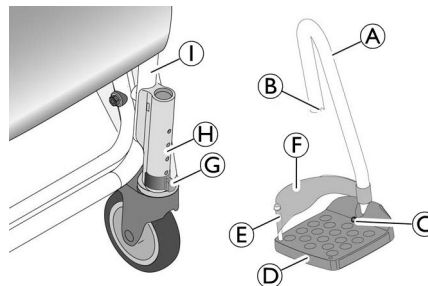1. Push the footrest **A** with the footplate **D** inwards into the guide **H** on the seat frame **I** until the groove **B** catches the pin on the clip **G**. The footrest can no longer be turned to the sides.

The assembly procedure is the same for the right and the left footrests.

## Adjusting the height of the footrests

1. Pull up the footrest **A** slightly.
2. Pull out the clip **G** and put it in again at the required position. Pull out the footrest slightly further if necessary.
3. Push down the footrest until the groove **B** catches the pin of the clip.
4. Set the other footrest to the same height as described.

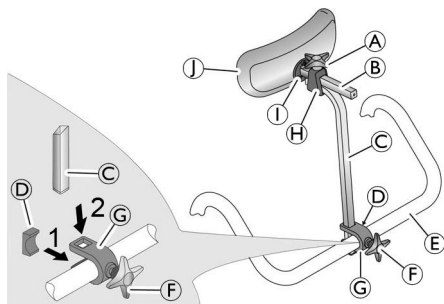
## Adjusting the inclination of the foot plates

### ! IMPORTANT!

- The inclination of the foot plates can be adjusted infinitely variably.
- Make sure, the right and left foot plate is adjusted in the same way.

1. Loosen the Allen screw © on the foot plate using a size 4 Allen key.
2. Turn the foot plate © to the desired position.
3. Tighten the Allen screw again.

## Fitting the headrest

1. Use the star handle © to insert the screw a little into the bracket ©.
2. Push the bracket with the screw onto the handle of the backrest © from behind in such a way that the screw engages in the hole of the backrest's handle.
3. Insert the plate © into the bracket from the front in such a way that both lugs of the plate face down and up and the curved plate side touches the handle of the backrest.

### ! IMPORTANT!

- On inserting the retainer, make sure that the curved side of the plate rests entirely on the handle of the backrest.

4. Push the grab handle © from above into the recesses of the bracket.
5. Use the star handle to tighten the screw until the grab handle is fixed.
6. Push the pad ① and the pad holder © into the holder on the grab handle ©.
7. Use the star handle © to tighten the upper screw until the pad holder is fixed.

To adapt it to the user, the grab handle can be fitted with the curve facing the front or the back.

The headrest pad ① can be removed and replaced by other pad shapes (optional). To remove it, loosen the retaining bolts ①.

## 5 Usage

### 5.1 Safety information

#### **WARNING!**

- Only use the shower and toilet commode on a flat surface and indoors.
- Observe the specifications on the identification label. Do not overload the shower and toilet commode.
- Observe the safety labels on the backside of the headrest and the side parts, 2.2 Labels and symbols on the product, page 7 .
- Do not place excessive weight on the shower and toilet commode.
- Always perform a visual inspection of the product for external damage before each use.

#### **WARNING!**

##### **Risk of slipping and rolling**

- Before use, make sure that the seat is securely fastened to the seat frame
- Lock all four casters using the red foot levers before sitting into the commode or standing up from it, as well as in the parking position.

#### **WARNING!**

##### **Risk of tipping**

- Swivel the footrests to the side or fold the footplates up before sitting down or standing up.
- Before sitting down or standing up, turn the casters so that the lock is pointing forwards.
- Never stand on the footrests.
- Only drive the user on the commode when the seat frame and backrest are fixed in the upright position.
- Never remove the pneumatic springs.

#### **WARNING!**

##### **Risk of tipping forwards!**

- Front antitippers (Art. No. 1535936) are available to prevent the shower commode chair from tipping forward.

#### **CAUTION!**

##### **Risk of catching due to tilt mechanism**

- When tilting the seat frame, make sure that the user's arms are placed on the armrests.

#### **IMPORTANT!**

##### **Risk of breaking the headrest**

- Only steer or push the shower and toilet commode using the handle on the backrest, never with the headrest.

## 5.2 Using the shower and toilet commode

### Swiveling the armrests

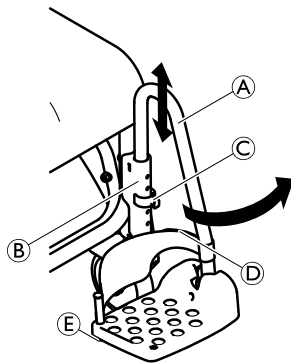
#### CAUTION!

**Risk of getting fingers caught in the folding mechanism**

- Do not insert fingers in the articulated joint when folding the armrests up or down.
- Do not insert fingers between the armrest and the backrest when folding back the armrests.
- Make sure that no parts get caught.

### Swiveling the footrests

#### Entry position/exit position:

1. Pull the footrest **A** in the guide **B** a little upwards, until the groove no longer engages in the pin of the clip **C**.
2. Swivel the footrest to the side.

#### Driving position:

#### CAUTION!

**Risk of accident due to loose footrest**

- Fix footrests in the driving position.

1. Turn the footrest **A** inwards.
2. Push the footrest downward, until the groove engages in the pin of the clip **C**.

### Removing/fitting the heel strap

1. Open the Velcro® strip **D** on the heel strap.
2. Remove the screw **E** from the underside of the footplate.
3. Remove the heel strap from the flexible plastic pin.

Attach the strap using the same procedure in the reverse order.

#### IMPORTANT!

- When installing ensure that the Velcro® strip of the heel strap is facing backwards.

## Swivelling the seat frame with backrest

### ! IMPORTANT!

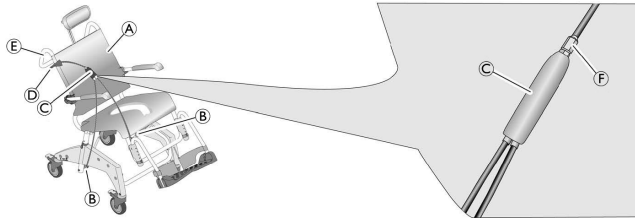
- Only use the removable handle and only swivel the seat frame with backrest if a user is sitting on the commode.
- On swiveling the seat frame with backrest, pull the backrest backwards or forwards using the handle.
- Never reach into the area between the seat frame with backrest and the front tubular frame between the side sections when swivelling.
- Do not drive when the seat frame of the backrest is swivelled.

## Swivelling forwards without a user

1. Stand next to the commode.
2. Pull the removable handle with one hand and keep in this position.
3. Press the seat frame with backrest down by pushing the middle of the seat surface with your other hand.
4. Release the removable handle once the seat frame with backrest is swivelled forwards. The seat frame is now fixed in this position.

## Swivelling backwards/forwards

 The seat frame can be tilted approx. 35° backwards together with the backrest.

1. Pull the removable handle **D** and hold it in this position. The distributor **C** is used to release the blocking of the pneumatic springs **B**.
2. Press the seat frame **A** at the handle of the backrest **E** backwards or forwards together with the backrest. The pneumatic springs **B** are extended or retracted.
3. Release the removable handle. The seat frame is fixed at the currently set inclination.

## Removing/fitting the seat plate

### Removing:

1. Pull the seat plate upwards on both sides.

### Fitting:

#### **CAUTION!**

#### **Risk of getting fingers caught**

- Do not insert fingers between the seat surface and the seat frame when pressing down on the seat plate.

1. Place the seat plate on the seat frame with the cutout to the front.
2. Push the seat plate backwards as far as it will go.
3. With your hand flat, press the seat plate down at the sides and then at the front and back until the seat plate snaps into the seat frame.

## Locking the position locking device

The position locking device only locks in position if the wheel is in forward position.

1. Lock the locking device at the caster with the blue foot pedal.  
The caster is fixed in forward position.

---

## 6 Transport

---

### 6.1 Safety information

The product can be dismantled for transport into the parts of the scope of delivery, ® 3.1 Overview, page 9.

1. Disassemble the product, ® 4.2 Assembling the shower and toilet commode, page 10 by following the assembly steps in reverse order.

#### **IMPORTANT!**

##### **Risk of damage to the product**

- Transport the product in a dry area at a temperature of between 0 °C and 40 °C, atmospheric pressure of 795 - 1060 hPa and a humidity of 30 - 75%.
- Do not expose the product to direct sunlight.

## 7 Maintenance

### 7.1 Safety information

The product is maintenance-free, provided the cleaning and safety instructions outlined above are adhered to.

#### **WARNING!**

- Check product regularly for damage and to ensure that the screws are firmly fitted and it is securely assembled.

#### **WARNING!**

- Hygiene is especially important as regards the toilet.
- Regularly clean the shower and toilet commode and keep it clean.

This product can be disinfected. Choose the disinfectant in accordance with Aquatec® care guidelines. These are available on request from Aquatec®, for contact details see back page of this user manual.

### 7.2 Cleaning and disinfection

The product can be cleaned and disinfected using commercially available agents.

#### **IMPORTANT!**

- In the sensitive area of the bathroom, hygiene is particularly important.
- Keep the product clean at all times and wash it regularly.

1. Disinfect the product by wiping down all generally accessible surfaces with disinfectant (after disassembling the product, if possible).

#### **IMPORTANT!**

- All cleaning agents and disinfectants used must be effective, compatible with one another and must protect the materials they are used to clean. For further information on decontamination in Healthcare Environments, please refer to 'The National Institute for Clinical Excellence' guidelines on Infection Control [www.nice.org.uk/CGI39](http://www.nice.org.uk/CGI39) and your local infection control policy.
- Clean by hand.
  - Clean without scouring agent.

### Cleaning in a domestic environment

#### **IMPORTANT!**

- Risk of damage to the product
- Do not disassemble the holders for the accessories.

1. Remove any accessories fitted.
2. Pull the footrests out of the guides.

3. Remove the seat plate and the backrest cover.
4. Remove the side parts.
5. Reinsert the clips in the seat frame immediately so that they are not misplaced.
6. Wash the individual parts using a cloth or brush and commercially available cleaning agents and disinfectants.
7. Rinse the parts with warm water.
8. Dry the parts with a cloth.
9. Re-assemble the product, 4.2 Assembling the shower and toilet commode, page 10.

### Cleaning in a clinical environment

-  In addition to cleaning as described above, clean the product after the release of each patient, after ending of the period of use or on a minimum monthly basis (depending on which occurs first) at 85 °C for 3 minutes in the autoclave.

#### **CAUTION!** **Risk of burning your skin**

- Allow the product to cool after cleaning in the autoclave.

### Cleaning the backrest cover (optional)

1. Remove the backrest cover.
2. Machine-wash the cover at max. 60 °C using a mild detergent.

#### **IMPORTANT!** **Risk of damage**

- Washing at higher temperatures can cause shrinkage.
- Do not tumble dry covers.

#### **IMPORTANT!**

- Excessive cleaning can damage textile.
- We recommend to change the backrest cover every two years.

## 7.3 Changing the castors

#### **WARNING!**

- Risk of accident due to incorrectly fitted castors.**
- The castors must be replaced by a specialist dealer.

---

## 8 After Use

---

### 8.1 Storage

**WARNING!**

**Risk of damage to the product**

- Store the product in a dry area at a temperature of between 0 °C and 40 °C, air pressure of 860 - 1060 hPa and humidity of 30 - 75%.
- Do not store the product near heat sources.
- Keep the product out of direct sunlight.

### 8.2 Re-use

The product is suitable for repeated use. The number of times it can be used depends on how often and in which way the product is used. Before reuse, clean and disinfect the product thoroughly (® 7.2 Cleaning and disinfection, page 20) and check the safety functions, and reset if necessary.

### 8.3 Disposal

The disposal and recycling of used devices and packaging must comply with the applicable legal regulations.

Please refer to the manufacturer for details of the proper and complete disposal.

## 9 Troubleshooting

### 9.1 Identifying and repairing faults

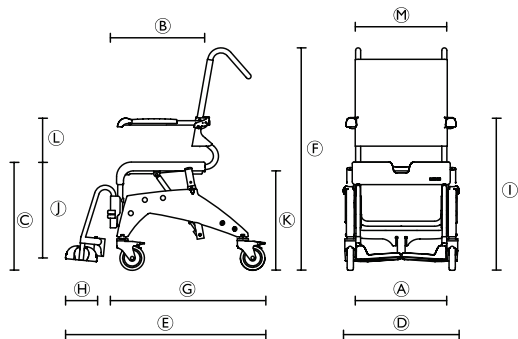
The following table provides information on malfunctions and their possible causes. If you cannot correct the malfunction using the remedies named, please contact your specialist dealer directly.

<b>Fault</b>	<b>Possible cause</b>	<b>Remedy</b>
Difficulty in adjusting the seat height	Dirt between the seat frame and the side section	Disassemble product into the parts of the scope of delivery and clean thoroughly
The casters are difficult or impossible to turn	Dirt between the fork and the rim	Thoroughly clean the casters
Difficulty pushing the footrests to the side	Deposits in the guides	Remove the footrests and thoroughly clean them
The seat frame with backrest cannot be swiveled	Nut loosened on the distributor	Tighten the nut
	The pneumatic is faulty	Contact your specialised dealer

## 10 Technical data

### 10.1 Dimensions and weight

Model	Ocean VIP	Ocean VIP XL	Ocean Dual VIP
Dimensions [mm] and weight [kg]			
A	500		
B	450		
C	510 – 610		
D	595	625	625
E	106		
F	1085 – 1185		
G	845		
H	160		

Model	Ocean VIP	Ocean VIP XL	Ocean Dual VIP
①	720 – 860		
②	345 – 465		
③	420 – 520 (with toilet pan holder) 440 – 540 (without toilet pan holder)		
④	215 / 245		
⑤	475	555	475
Width, hygiene recess	205		
Depth, hygiene recess	320		
Angle of inclination seat	0° – 35°		
Angle of inclination backrest	-	-	10° – 50°
Caster diameter	5" (127 mm)		
Product weight	23.5	23.5	28.3
Minimum / Maximum User weight	40 / 150		

## 10.2 Materials

Frame	Stainless steel, powder-coated
Seat plate	Polypropylene
Backrest cover	Nylon, PVC-coated
Footrests	Polypropylene, fiber-glass reinforced
Armrests	Polypropylene
Castors	Plastics and stainless steel

All product components are corrosion resistant and latex free.

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Invacare AG
Invacare Ireland Ltd,
Aquatec Operations GmbH