# Invacare® H430A/2

EN	<b>Toilet Seat Raiser</b> User Manual . . . . .	1
DE	<b>Toilettensitzerhöhung</b> Gebrauchsanweisung . . . . .	3
FR	<b>Rehausse de cuvette de toilettes</b> Manuel d'utilisation . . . . .	6
IT	<b>Rialzo per tavoletta del WC</b> Manuale d'uso . . . . .	8
NL	<b>Verhoging voor toiletzitting</b> Gebruiksaanwijzing . . . . .	10
ET	<b>WC-poti kõrgendus</b> Kasutusjuhend . . . . .	13
LV	<b>Tualetes poda sēdvirsmas paaugstinājums</b> Instrukciju rokasgrāmata . . . . .	15
LT	<b>Tualetto sėdynės pakėliklis</b> Naudojimosi instrukcija . . . . .	18
HR	<b>Podizač WC sjedala</b> Upute za uporabu . . . . .	20
RO	<b>Înălțător pentru scaunul de toaletă</b> Instrucțiuni de folosire . . . . .	22
BG	<b>Устройство за повдигане за седалка на тоалетна чиния</b> РЪКОВОДСТВО . . . . .	24

EN

## I General

### I.1 Symbols in this user manual

In this user manual, warnings are marked with symbols. The title next to the warning symbol indicates the level of danger.

#### WARNING

Indicates a dangerous situation which could lead to death or serious injury if not avoided.

#### CAUTION

Indicates a dangerous situation which could lead to minor injuries if not avoided.

#### IMPORTANT

Indicates a dangerous situation which could lead to damage if not avoided.

Useful tips, recommendations and information to ensure efficient and smooth operation.

This product complies with the directive 93/42/EEC for medical products. The launch date for this product is specified in the CE declaration of conformity.

Manufacturer

This manual **MUST** be given to the user of the product. **BEFORE** using this product, read this manual and save for future reference.

### I.2 Intended use

The product serves exclusively to raise the toilet seat and is therefore only intended for installation on toilets in an indoor environment. Any other use is prohibited.

### I.3 Scope of delivery

- Toilet seat raiser with toilet seat and lid
- Armrest (2x)
- User Manual

### I.4 Warranty

We provide a manufacturer's warranty for the product in accordance with our General Terms and Conditions of Business in the respective countries. Warranty claims can only be made through the provider from whom the appliance was obtained.

Yes, you can.®

## Quality

Quality is of the utmost importance for our company; all processes are based on the ISO 9001 and ISO 13485 standards.

This product features the CE mark, in compliance with the Medical Device Directive 93/42/EEC Class I.

The product meets the requirements of the standards DIN EN 12182 (Technical aids for disabled persons - general requirements and test methods), DIN EN ISO 14971 (Medical devices - Application of risk management to medical devices) and DIN EN ISO 10993-1 (Biological evaluation of medical devices - Part 1: Evaluation and testing within a risk management system).

Invacare® is continuously working towards ensuring that the company's impact on the environment, locally and globally, is reduced to a minimum. It is Invacare's goal to ensure that we, at the widest possible range, use REACH compliant materials and components.

For further information please contact Invacare® in your country (for addresses, see back page of this manual).

## 1.5 Service life

The expected service life of this product is three years when used in accordance with safety instructions and intended use, stated in this manual. The effective service life can vary according to frequency and intensity of use.

## 2 Safety

### 2.1 Safety information

**WARNING!**

**Risk of falling!**

- Do not stand on this product.
- Do not use as a climbing aid.

**WARNING!**

**Risk of falling!**

- There is a risk that the armrest will break, if pressure is applied on one armrest only due to overloading.
- When sitting down to or standing up from the toilet seat raiser, always use both armrests at the same time.


**WARNING!**

**Risk of injury!**

- Do not use if defective.
- Do not make any unauthorized alterations or modifications to the product.

### 2.2 Labels and symbols on the product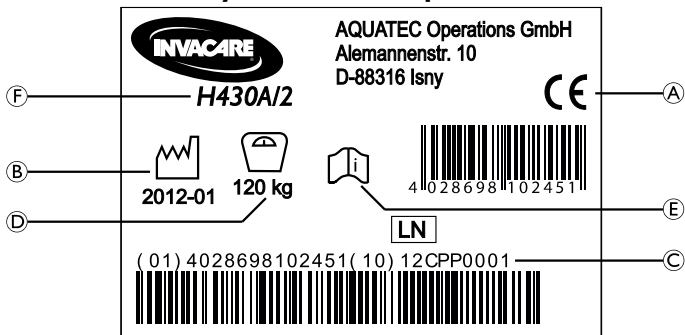

(A)	Conformity mark
(B)	Production date
(C)	Serial number
(D)	Load capacity
(E)	Note (observe accompanying documents)
(F)	Product designation


The identification label is attached to the backside of the toilet seat raiser.

## 3 Setup

### 3.1 Installing the product

1. Remove the cups (A) from the hexagon nuts on both sides.
2. Using a spanner (SW14), remove the hexagon nuts from the front horizontal bar (C) on both sides.
3. Remove the star grip screws (B) from the rear horizontal bar (D) on both sides.
4. Place the front and rear horizontal bar into the desired position, see 3.2 Adjusting the seat height and seat angle, page 2.
5. Re-attach the hexagon nuts and cups (A) on both sides.
6. Re-attach the star head screw (B) on both sides.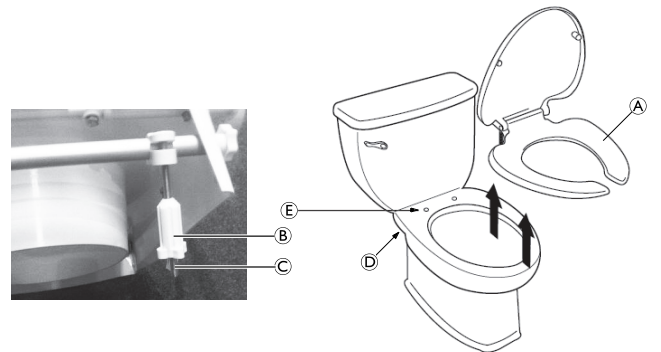

7. Remove the existing toilet lid (A).
8. Remove the plastic fixation nuts (B) from the threaded pins (C).
9. Insert the threaded pins (C) into the fixing holes (E) of the toilet bowl.
10. Fasten the threaded pins with the plastic fixation nuts from the underside of the toilet bowl (D).

**WARNING!**


**Risk of falling!**

- Make sure that the star grip of both plastic fixation nuts (B) points downwards, as shown in the figure above.

11. Push the armrests (E) sideways into the rear holes of the toilet seat raiser body until they audibly engage.
12. Fold the armrests down until they engage in the holders (F).

### 3.2 Adjusting the seat height and seat angle

Following seat heights and seat angles can be set:

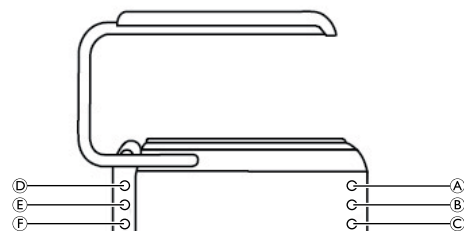Combination of positions				
Front seat height		Rear seat height		Seat angle
A	60 mm	D	60 mm	0°
		E	100 mm	4°, sloping to the front
		F	140 mm	8°, sloping to the front
B	100 mm	E	100 mm	0°
		D	60 mm	4°, sloping to the rear
		F	140 mm	4°, sloping to the front
C	140 mm	F	140 mm	0°
		E	100 mm	4°, sloping to the rear
		D	60 mm	8°, sloping to the rear

## 4 Maintenance

### 4.1 Maintenance and periodic inspection

The product is maintenance-free, provided the cleaning and safety instructions outlined in this user manual are adhered to.

#### WARNING!

- Before first use, clean the product thoroughly.
- Before every use, check the product for damage and ensure that it is securely assembled.

### 4.2 Cleaning and disinfection

#### IMPORTANT!

All cleaning agents and disinfectants used must be effective, compatible with one another and must protect the materials they are used to clean. For further information on decontamination in Health care Environments, please refer to 'The National Institute for Clinical Excellence' guidelines on Infection Control [www.nice.org.uk/CGI39](http://www.nice.org.uk/CGI39) and your local infection control policy.

- Clean by hand.
- Do not use any aggressive (pH value under 4.5 or over 8.5) or abrasive cleaning agents.
- Max. cleaning temperature 60 °C.

The product can be cleaned and disinfected using commercially available agents.

- Disinfect the product by wiping down all generally accessible surfaces with disinfectant (after disassembling the product, if possible).

## 5 After Use

### 5.1 Re-use


The product is suitable for repeated use. The number of times it can be used depends on how often and in which way the product is used. Before reuse, clean and disinfect the product thoroughly (© 4.2 Cleaning and disinfection, page 3), check the safety functions and reset if necessary.

### 5.2 Disposal

The disposal and recycling of used devices and packaging must comply with the applicable local legal regulations.

## 6 Technical data

### 6.1 Dimensions and weight

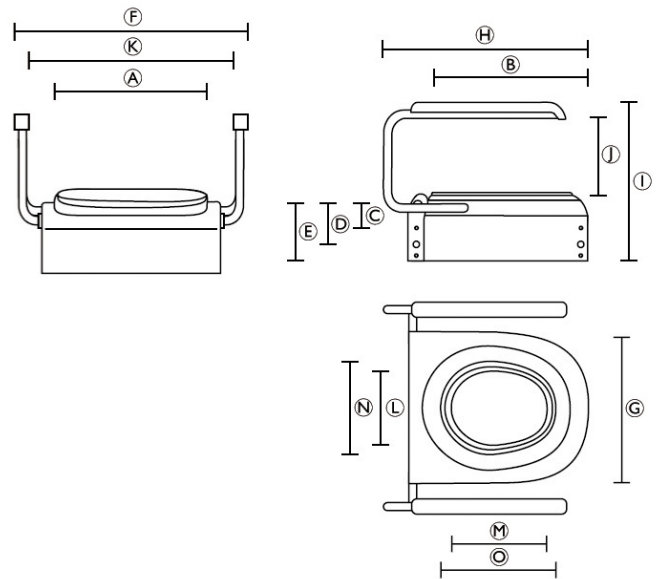A	360 mm	I	355 mm
B	390 mm	J	200 mm
C	60 mm	K	470 mm
D	100 mm	L	230 mm
E	140 mm	M	235 mm
F	570 mm	N	260 mm
G	395 mm	O	300 mm
H	480 mm		

Product weight: 6.2 kg

Max. user weight: 120 kg

Operating temperature: 10 – 40 °C

### 6.2 Materials

Body toilet seat raiser	PP
Toilet seat with lid	PP
Armrest tubes	Q345
Armrest pads	PU

All product components are corrosion resistant and latex free.

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Invacare Austria GmbH
Invacare GmbH, EDO
Aquatec Operations GmbH