## Invacare® Aqua / Ocea

HI40, HI41, HI42, HI43, HI44, HI45, HI46 / H320

en	<b>Bathroom handle</b> User Manual . . . . .	1
de	<b>Badezimmergriff</b> Gebrauchsanweisung . . . . .	4
fr	<b>Poignée pour salle de bains</b> Manuel d'utilisation . . . . .	6
it	<b>Maniglione da bagno</b> Manuale d'uso . . . . .	7
es	<b>Asidero de cuarto de baño</b> Manual del usuario . . . . .	9
pt	<b>Pega para casa de banho</b> Manual de utilização . . . . .	10
nl	<b>Badkamerhandvat</b> Gebruiksaanwijzing . . . . .	12
sv	<b>Badrumshandtag</b> Bruksanvisning . . . . .	13
no	<b>Baderomshåndtak</b> Bruksanvisning . . . . .	15
da	<b>Badeværelsesgreb</b> Brugsanvisning . . . . .	16
fi	<b>Kylpykahva</b> Käyttöohje . . . . .	17

This manual MUST be given to the user of the product.  
BEFORE using this product, read this manual and save for future reference.

Yes, you can.®

# I General

## I.1 Images

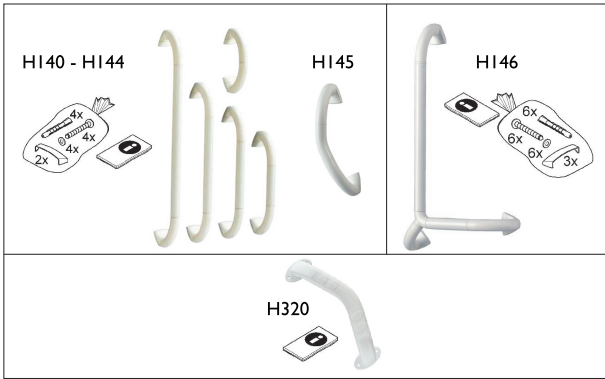
Fig. 1

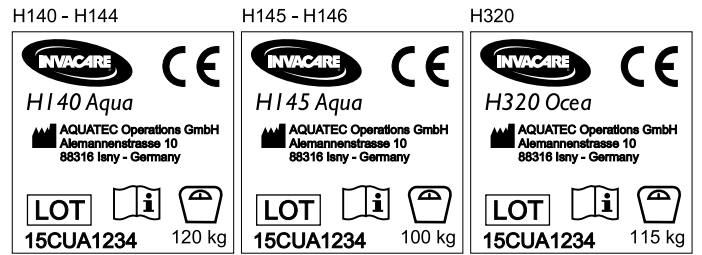
Fig. 2

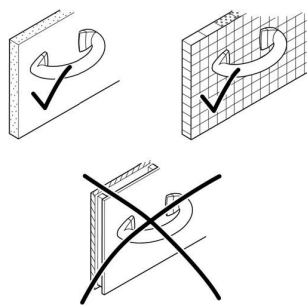
Fig. 3

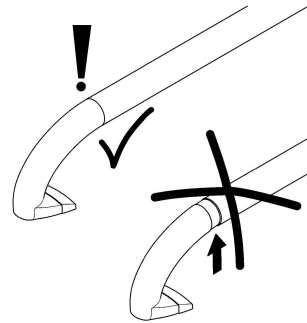
Fig. 4

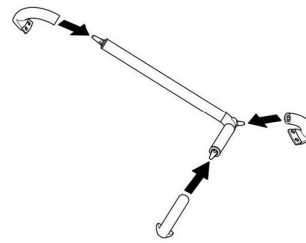
Fig. 5

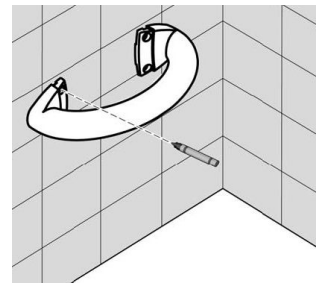
Fig. 6

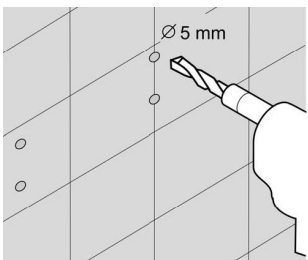
Fig. 7

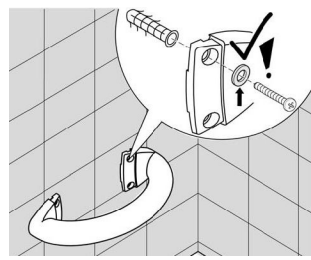
Fig. 8

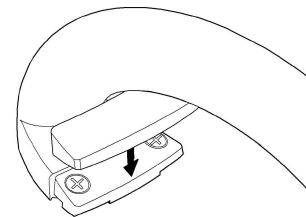
Fig. 9

Fig. 10

Fig. 11

## 1.2 Symbols in this user manual

In this user manual, warnings are marked with symbols. The title next to the warning symbol indicates the level of danger.

### WARNING

Indicates a dangerous situation which could lead to death or serious injury if not avoided.

### IMPORTANT

Indicates a dangerous situation which could lead to damage if not avoided.

Useful tips, recommendations and information to ensure efficient and smooth operation.

This product complies with the directive 93/42/EEC for medical products. The launch date for this product is specified in the CE declaration of conformity.

Manufacturer

## 1.3 Intended use

The bathroom handles were designed to support natural movement and not to bear the full body weight of the user. Any other use is prohibited.

## 1.4 Scope of delivery (Fig. 1)

- Bathroom handle
- Wall mounting kit (screws, washers, wall plugs, covering caps) \*
- User manual

\* Not for H320 Ocea

## 1.5 Warranty

We provide a manufacturer's warranty for the product in accordance with our General Terms and Conditions of Business in the respective countries. Warranty claims can only be made through the provider from whom the appliance was obtained.

## 1.6 Standards and regulations

Quality is of the utmost importance for our company; all processes are based on the ISO 9001 and ISO 13485 standards.

This product features the CE mark, in compliance with the Medical Device Directive 93/42/EEC Class I.

Invacare® is continuously working towards ensuring that the company's impact on the environment, locally and globally, is reduced to a minimum. We only use REACH compliant materials and components.

For further information please contact Invacare® in your country (for addresses, see back page of this manual).

## 1.7 Service life

The expected service life of this product is five years when used daily in accordance with safety instructions and intended use, stated in this manual. The effective service life can vary according to frequency and intensity of use.

# 2 Safety

## 2.1 Safety information

### WARNING!

#### Risk of falling!

- Do not use if defective.
- Do not use as sports equipment.
- Do not use as a climbing aid.

## 2.2 Labels and symbols on the product

### Identification label (Fig. 2)

	Conformity mark		Maximum safe load
	Lot number		Read user manual
	Manufacturer		

# 3 Setup and usage

## 3.1 Installing the bathroom handle

### WARNING!

#### Risk of falling!

- Mount only to walls that have a sufficient load-bearing capacity (Fig. 3).
- Only for H146: Make sure that the wall-fixing elements are fully inserted into the handle bar (Fig.4).

1. Only for H146: Attach the wall-mounting elements to the handle bar (Fig. 5).
2. Place the bathroom handle in the desired position against the wall and mark the mounting points using a scribing tool (Fig. 6).
3. Use a drill to make suitably sized holes at the points marked on the wall (Fig. 7).
4. Fix the bathroom handle to the wall using the included wall mounting kit (Fig. 8).

A wall mounting kit is not included for model H320.

### WARNING!

#### Risk of injury caused by a fall

- Only use suitable wall plugs and bolts for mounting the H320.

5. Only for H140 – H146: Place the screw covering caps on the fixing elements (Fig. 9).

## 3.2 Using the bathroom handle

### WARNING!

#### Risk of falling or slipping!

- Before each use, check that the bathroom handle is securely fastened and sits firmly.

1. Grip the handle of the bathroom handle with one hand and carry out the desired body movement (Fig. 10 & 11).

# 4 Maintenance

## 4.1 Maintenance and periodic inspection

The product is maintenance-free, provided the cleaning and safety instructions outlined in this user manual are adhered to.

### WARNING!

- Before first use, clean the product thoroughly.
- Before every use, check the product for damage and ensure that it is securely assembled.

## 4.2 Cleaning and disinfection

### IMPORTANT!

All cleaning agents and disinfectants used must be effective, compatible with one another and must protect the materials they are used to clean. For further information on decontamination in Health Care Environments, please refer to 'The National Institute for Clinical Excellence' guidelines on Infection Control [www.nice.org.uk/CG139](http://www.nice.org.uk/CG139) and your local infection control policy.

- Clean by hand.
- Clean without scouring agents.
- Max. cleaning temperature 65 °C for 3 minutes.

The product can be cleaned and disinfected using commercially available agents.

- I. Disinfect the product by wiping down all generally accessible surfaces with disinfectant.

## 4.3 Re-use

The product is suitable for repeated use. The number of times it can be used depends on how often and in which way the product is used. Before reuse, clean and disinfect the product thoroughly.

### WARNING!

#### Risk of injury

Worn fastening elements can suddenly come loose and cause you to fall.

- Always use new wall plugs and bolts when repositioning the product.

## 4.4 Disposal

The disposal and recycling of used devices and packaging must comply with the applicable local legal regulations.

## 5 Technical data

Product	Length [mm]	Max. load force	Product weight [kg]	Materials*
H140	250	120 kg	0.3	ABS
H141	400		0.5	
H142	500		0.6	
H143	600		0.7	
H144	800	100 kg	0.9	Aluminum / ABS
H145	400 (angled 150°)		0.4	
H146	700/400		0.98	
H320	330 (angled 135°)	115 kg	0.18	PP

The tube diameter of H140 – H146 is 350 mm.

\*All product components are corrosion resistant and Latex free.

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## Environmental parameters

Operating conditions	
Ambient temperature	10 - 40 °C
Relative humidity	15% - 93%, non-condensing
Atmospheric pressure	940 - 1060 hPa
Storage and shipping conditions	
Ambient temperature	0 - 40 °C
Relative humidity	30% - 75%, non-condensing
Atmospheric pressure	795 - 1060 hPa

Invacare® distributors

### Belgium & Luxemburg:

Invacare nv  
Autobaan 22  
B-8210 Loppem  
Tel: (32) (0)50 83 10 10  
Fax: (32) (0)50 83 10 11  
belgium@invacare.com  
www.invacare.be

### Danmark:

Invacare A/S  
Sdr. Ringvej 37  
DK-2605 Brøndby  
Tel: (45) (0)36 90 00 00  
Fax: (45) (0)36 90 00 01  
denmark@invacare.com  
www.invacare.dk

### Deutschland:

Invacare GmbH,  
Alemannenstraße 10  
D-88316 Isny  
Tel: (49) (0)7562 700 0  
Fax: (49) (0)7562 700 66  
kontakt@invacare.com  
www.invacare.de

### España:

Invacare SA  
c/Areny s/n, Polígon Industrial de  
Celrà  
E-17460 Celrà (Girona)  
Tel: (34) (0)972 49 32 00  
Fax: (34) (0)972 49 32 20  
contactsp@invacare.com  
www.invacare.es

### France:

Invacare Poirier SAS  
Route de St Roch  
F-37230 Fondettes  
Tel: (33) (0)2 47 62 64 66  
Fax: (33) (0)2 47 42 12 24  
contactfr@invacare.com  
www.invacare.fr

### Ireland:

Invacare Ireland Ltd,  
Unit 5 Seatown Business Campus  
Seatown Road, Swords, County Dublin  
Tel: (353) 1 810 7084  
Fax: (353) 1 810 7085  
ireland@invacare.com  
www.invacare.ie

### Italia:

Invacare Mecc San s.r.l.,  
Via dei Pini 62,  
I-36016 Thiene (VI)  
Tel: (39) 0445 38 00 59  
Fax: (39) 0445 38 00 34  
italia@invacare.com  
www.invacare.it

### Nederland:

Invacare BV  
Galvanistraat 14-3  
NL-6716 AE Ede  
Tel: (31) (0)318 695 757  
Fax: (31) (0)318 695 758  
nederland@invacare.com  
www.invacare.nl

### Norge:

Invacare AS  
Grensesvingen 9, Postboks 6230, Etterstad  
N-0603 Oslo  
Tel: (47) (0)22 57 95 00  
Fax: (47) (0)22 57 95 01  
norway@invacare.com  
www.invacare.no

### Österreich:

Invacare Austria GmbH  
Herzog Odilostrasse 101  
A-5310 Mondsee  
Tel: (43) 6232 5535 0  
Fax: (43) 6232 5535 4  
info-austria@invacare.com  
www.invacare.at

### Portugal:

Invacare Lda  
Rua Estrada Velha, 949  
P-4465-784 Leça do Balio  
Tel: (351) (0)225 1059 46/47  
Fax: (351) (0)225 1057 39  
portugal@invacare.com  
www.invacare.pt

### Sverige:

Invacare AB  
Fagerstagatan 9  
S-163 53 Spånga  
Tel: (46) (0)8 761 70 90  
Fax: (46) (0)8 761 81 08  
sweden@invacare.com  
www.invacare.se

### Suomi:

Algol-Trehab Oy  
Karapellontie 6, 02610 Espoo  
Puhelin 09 50991  
Faksi 09 5099490  
Sähköposti info.algoltrehab.fi@algol.fi  
Internet www.algoltrehab.fi

### Schweiz / Suisse / Svizzera:

Invacare AG  
Benkenstrasse 260  
CH-4108 Witterswil  
Tel: (41) (0)61 487 70 80  
Fax: (41) (0)61 487 70 81  
switzerland@invacare.com  
www.invacare.ch

### United Kingdom:

Invacare Limited  
Pencoed Technology Park, Pencoed  
Bridgend CF35 5AQ  
Tel: (44) (0) 1656 776 222  
Fax: (44) (0) 1656 776 220  
uk@invacare.com  
www.invacare.co.uk

### USA and Canada:

Clarke Health Care Products  
7830 Steubenville Pike  
Oakdale, PA 15071, USA  
Toll Free: (888) 347-4537  
Phone: (724) 695-2122  
email: info@clarkehealthcare.com  
www.clarkehealthcare.com

### Canada:

Invacare Canada LP  
570 Matheson Blvd E. Unit 8  
Mississauga Ontario  
L4Z 4G4, Canada  
Phone: (905) 890 8300  
Fax: (905) 501 4336

Aquatec Operations GmbH  
Alemannenstrasse 10  
D-88316 Isny