# Invacare® Stileo

H170 / H171 / H172


en	Suction Grab Bar User Manual .....	2
de	Saughaltegriff Gebrauchsanweisung .....	5
fr	Poignée de maintien à ventouses Manuel d'utilisation .....	9
it	Barra di supporto a ventosa Manuale d'uso .....	12
es	Barra de apoyo con ventosa Manual del usuario .....	15
pt	Barra de apoio com ventosas Manual de utilização .....	19
nl	Handgreep met zuignappen Gebruiksaanwijzing .....	22
no	Sugekopphåndtak Bruksanvisning .....	25
da	Gribehåndtag med sugekop Brugsanvisning .....	29
sv	Suggriphandtag Bruksanvisning .....	32
fi	Imukuppikahva Käyttöohje .....	35

This manual MUST be given to the user of the product.  
BEFORE using this product, this manual MUST be read and saved for future reference.

**Yes, you can.®**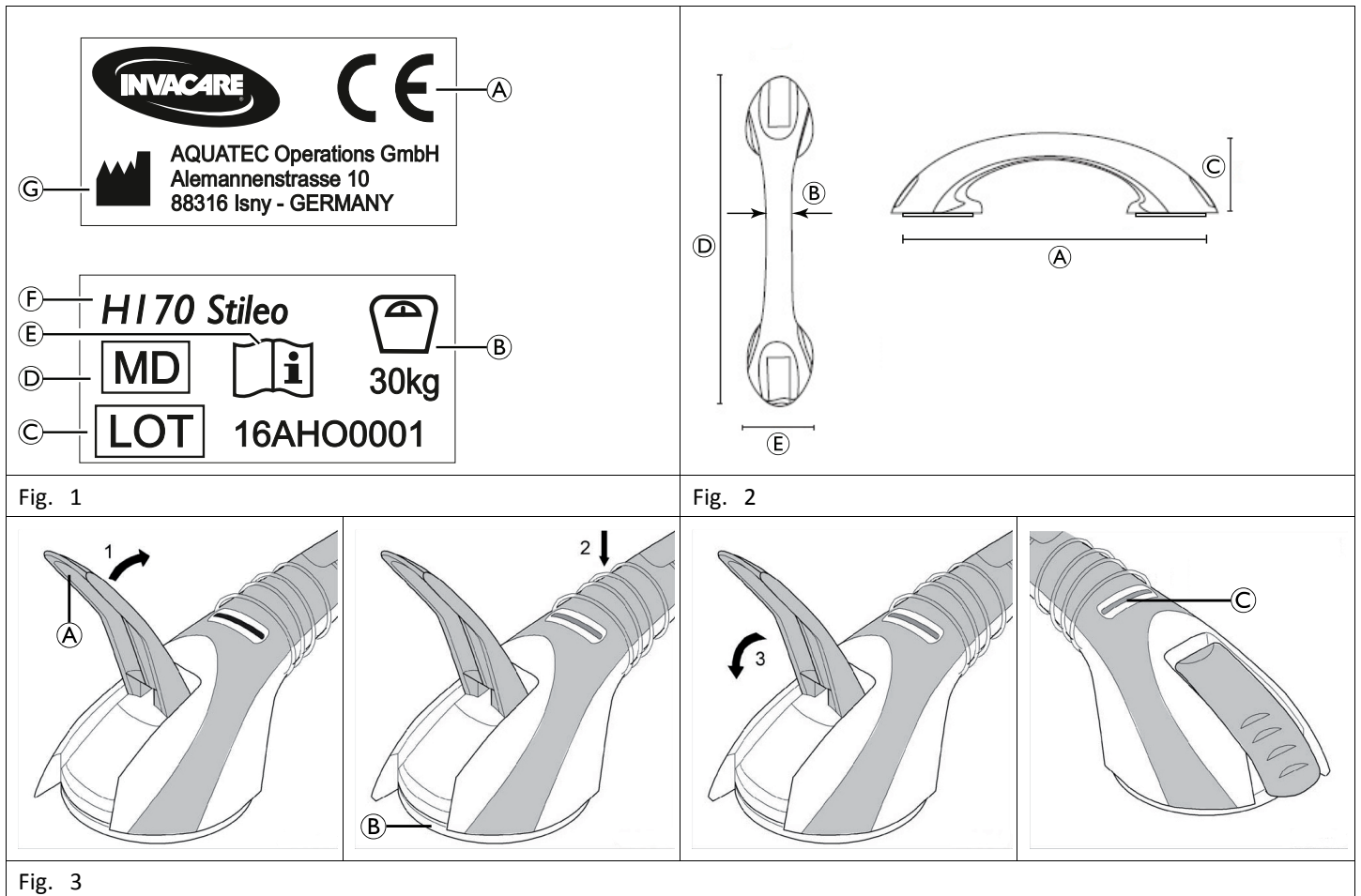

**en**

## 1 Introduction

This user manual contains important information about the handling of the product. To ensure safety when using the product, read the user manual carefully and follow the safety instructions.

Invacare reserves the right to alter product specifications without further notice.

Before reading this document, make sure you have the latest version. You find the latest version as a PDF on the Invacare website.

If you find that the font size in the printed document is difficult to read, you can download the PDF version from the website. The PDF can then be scaled on screen to a font size that is more comfortable for you.

In case of a serious incident with the product, you should inform the manufacturer and the competent authority in your country.

## 2 Warranty Information

We provide a manufacturer's warranty for the product in accordance with our General Terms and Conditions of Business in the respective countries.

Warranty claims can only be made through the provider from whom the product was obtained.

## 3 Compliance

Quality is fundamental to the company's operation, working within the disciplines of ISO 13485.

This product features the CE mark, in compliance with the Medical Device Regulation 2017/745 Class I. The launch date of this product is stated in the CE declaration of conformity.

We are continuously working towards ensuring that the company's impact on the environment, locally and globally, is reduced to a minimum.

We only use REACH compliant materials and components.

### 3.1 Product-specific Standards

H171 and H172 have been tested and conform to EN 12182 (Assistive products for persons with disability - General requirements and test methods) and related standards. H170 has been tested and conforms to EN 17966 (Assistive products for personal hygiene that support users - Requirements and test methods) and related standards.

For further information about local standards and regulations, contact your local Invacare representative. See addresses at the end of this document.

## 4 Service Life

The expected service life of this product is five years when used daily and in accordance with the safety instructions, maintenance intervals and correct use, stated in this manual. The effective service life can vary according to frequency and intensity of use.

## 5 Limitation of Liability

Invacare accepts no liability for damage arising from:

- Non-compliance with the user manual
- Incorrect use
- Natural wear and tear
- Incorrect assembly or set-up by the purchaser or a third party

- Technical modifications
- Unauthorised modifications and/or use of unsuitable spare parts

## 6 Safety

### 6.1 Safety Information

#### WARNING!

##### Risk of Serious Injury or Damage

Improper use of this product may cause injury or damage.

- If you are unable to understand the warnings, cautions or instructions, contact a health care professional or provider before attempting to use this equipment.
- Do not use this product or any available optional equipment without first completely reading and understanding these instructions and any additional instructional material such as user manual, service manual or instruction sheet supplied with this product or optional equipment.

#### WARNING!

##### Risk of Falling!

On unsuitable surfaces, the suction grab bar may not achieve the required suction force and may as a result suddenly be detached from the surface.

- Only use the suction grab bar on smooth, non-porous and firm surfaces, which are clean and dry, steam-proof and imperviously sealed.
- Never attach a suction grab bar on a joint.

#### WARNING!

##### Risk of Falling!

- Do not use if defective.

#### WARNING!

##### Risk of Falling!

Through ageing, incorrect installation or as a result of a dry construction method, it is possible that the adhesive force of tiles and surface materials may be insufficient. For this reason, the prepared surface may become detached if load is applied to the correctly mounted suction grab bar.

- Check the stability of the prepared surface before using the suction grab bar for the first time.

#### WARNING!

##### Risk of Falling!

Different ambient conditions can reduce the suction force.

- Only use the suction grab bar at temperatures below 40 °C and up to a maximum altitude of 1000 m above sea level.
- Do not expose the suction grab bar to direct sunlight for prolonged periods.
- Never attach a suction grab bar to a glass surface.

#### WARNING!

##### Risk of Falling!

Excessive or intermittent loads can lead to spontaneous loss of suction force.

- Before using the suction grab bar, check the suitability for the relevant user and his/her usage behaviour.
- The maximum load that can be applied to the suction grab bar is 30 kg.

#### CAUTION!

##### Risk of Injury

Non-original or wrong parts may affect the function and safety of the product.

- Only use original parts for the product in use.
- Due to regional differences, refer to your local Invacare catalog or website for available options or contact your Invacare distributor. See addresses at the end of this document.

### 6.2 Labels and Symbols on the Product

#### Identification Label (Fig. 1)

Ⓐ	European conformity
Ⓑ	Maximum safe load
Ⓒ	Lot number
Ⓓ	Medical device
Ⓔ	Read user manual
Ⓕ	Product designation
Ⓖ	Manufacturer

#### Warning Label

Warning label for safe usage. See hazard statements in chapter 10.1 *Installing the Suction Grab Bar (Fig. 3), page 4.*

## 7 Intended Use

This product is intended to be used as a transfer aid (e.g.: to enter into or get out of a bathtub).

Intended users are any adults and adolescents with restricted or limited mobility (e.g.: legs and/or hips) or limited muscle strength. If used without an assistant, the users must be able to keep their own balance.

#### Indications

- Restricted or limited mobility (e.g.: legs and/or hips)
- Balance issues to stand up or walk due to transitional or permanent disabilities.

There are no contraindications known when using the product as intended.

The product is capable to support a local weight of 30 kg.

## 8 Product Description

Aquatec® Stileo is a suction grab bar (300 mm, 450 mm or 550 mm length) working with two suction cups ensuring the fixation of the bar on a flat and clean surface thanks to vacuum force. There are two strength indicators providing information about remaining strength of the suction cups ensuring the support.

## 9 Scope of Delivery

- Suction grab bar

- User manual

## 10 Setup and Usage

### 10.1 Installing the Suction Grab Bar (Fig. 3)

The suction grab bar can be installed on sealed and smooth surfaces.

**WARNING!**  
**Risk of Falling!**

Incorrect mounting may lead to the suction grab bar suddenly detaching from the surface. The check display on the suction grab bar allows the user to check if it is mounted correctly (it is not a vacuum display!).

- Before installation ensure that the suction grab bar is undamaged, free of oil, dust and dirt.

1. Release both folding levers Ⓐ.
2. Press and hold the suction grab bar evenly with both suction pads Ⓑ against a suitable surface.
3. Close both folding levers.

**WARNING!**  
**Risk of Falling!**

There are two displays © present on the suction grab bar indicating if the grab bar is installed correctly and stable.

- If one or both indicators display red, the suction grab bar is not safely installed and must be repositioned.
- Make sure that the indicators on the suction grab bar both display green before using the product.

Observe warning label regarding the indicators:

	<p>Red hand: Do not use the product when one or both indicators display red.</p> <p>Green check: Only use the product when both indicators display green.</p>
--	---

### 10.2 Removing the Suction Grab Bar (Fig. 3)

1. Release both folding levers Ⓐ.
2. Remove the suction grab bar from the surface.

Lift the suction pads Ⓑ for easier removal (e. g. with a finger).

### 10.3 Using the Suction Grab Bar

As the following safety information does not encompass all possible circumstances, it is up to the user in the individual case to check before use whether the suction grab bar is suitable for the intended application, under the existing conditions.

**WARNING!**

- Before every use, check the product for damage and ensure that it is securely mounted.

**WARNING!**

**Risk of Falling!**

If the suction grab bar is used for any use other than that intended, e.g. as a sports device for “chin-ups” or as a climbing aid, it can suddenly become detached.

- Only use the suction grab bar for its intended use.
- Do not use as sports equipment.
- Do not use as a climbing aid.
- Do not use as carrying handle for loads.

1. Grip the handle of the suction grab bar with one hand and carry out the desired body movement.

## 11 Maintenance

### 11.1 General Maintenance Information

This product is maintenance-free, provided that the cleaning and disinfection instructions are adhered to.

### 11.2 Cleaning and Disinfection

#### General Safety Information

**CAUTION!**

**Risk of Contamination**

- Take precautions for yourself and use appropriate protective equipment.

**IMPORTANT!**

Wrong fluids or methods can harm or damage the product.

- All cleaning agents and disinfectants used must be effective, compatible with one another and must protect the materials they are used to clean.
- Never use corrosive fluids (alkalines, acid etc.) or abrasive cleaning agents. We recommend an ordinary household cleaning agent such as dishwashing liquid, if not specified otherwise in the cleaning instructions.
- Never use a solvent (cellulose thinner, acetone etc.) that changes the structure of the plastic or dissolves the attached labels.
- Always make sure that the product is completely dried before taking it into use again.

For cleaning and disinfection in clinical or long-term care environments, follow your in-house procedures.

#### Cleaning Intervals

**IMPORTANT!**

Regular cleaning and disinfection enhances smooth operation, increases the service life and prevents contamination.

- Clean and disinfect the product regularly while it is in use,
- before and after any service procedure,
- when it has been in contact with any body fluids,
- before using it for a new user.

#### Cleaning

**IMPORTANT!**

- Clean by hand
- Maximum cleaning temperature 40 °C

1. Clean the product by wiping down all generally accessible surfaces.
2. Dry the parts with a cloth.

### Disinfection Instructions

 Information on recommended disinfectants and methods can be found on <https://vah-online.de/en/for-users>.

### In Domestic Care

Method: Follow the application notes for the used disinfectant and wipe-disinfect all accessible surfaces.

Disinfectant: Ordinary household disinfectant.

Drying: Allow the product to air-dry.

### In Clinical and Long Term Care

Follow your in-house disinfection procedures and only use the disinfectants and methods specified therein.

## 12 Reconditioning

This product is suitable for reuse. To recondition the product for a new user, carry out the following actions:

- Inspection
- Cleaning and disinfection

For detailed information, see *11 Maintenance, page 4*.

Make sure that the user manual is handed over with the product.

If any damage or malfunction is detected, do not reuse the product.

## 13 Disposal

Be environmentally responsible and recycle this product through your recycling facility at its end of life.

Disassemble the product and its components, so the different materials can be separated and recycled individually.

The disposal and recycling of used products and packaging must comply with the laws and regulations for waste handling in each country. Contact your local waste management company for information.

## 14 Technical Data (Fig. 2)

	Stileo H170	Stileo H171	Stileo H172
Ⓐ	290 mm	400 mm	500 mm
Ⓑ	35 mm		
Ⓒ	72 mm	75 mm	
Ⓓ	300 mm	450 mm	550 mm
Ⓔ	95 mm	100 mm	
∅ Suction pads	80 mm	93 mm	
Product weight	0.35 kg	0.5 kg	0.6 kg
Max. load force	30 kg		
Materials*	Grab bar: ABS / Suction pads: PVC		

\*All product components are corrosion resistant and do not contain natural rubber latex.

## 14.1 Environmental Parameters

Operating Conditions	
Ambient temperature	10 - 40 °C
Relative humidity	15% - 93%, non-condensing
Storage and Shipping Conditions	
Ambient temperature	0 - 40 °C
Relative humidity	30% - 75%, non-condensing

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Invacare Limited