SMOOV


SMOOV one O10

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Important safety instructions

- DE** | Wichtige Sicherheitshinweise
- EN** | Important safety instructions
- FR** | Consignes de sécurité importantes
- ES** | Indicaciones de seguridad importantes
- IT** | Importanti avvertenze per la sicurezza
- DK** | Sikkerhedshenvisninger
- NL** | Viktige sikkerhetsinstruksjoner
- NO** | Sikkerhetsinstruksjoner
- SE** | Säkerhetsanvisningar
- FI** | Turvallisuusohjeet
- PT** | Instruções de segurança
- PL** | Instrukcje dotyczące bezpieczeństwa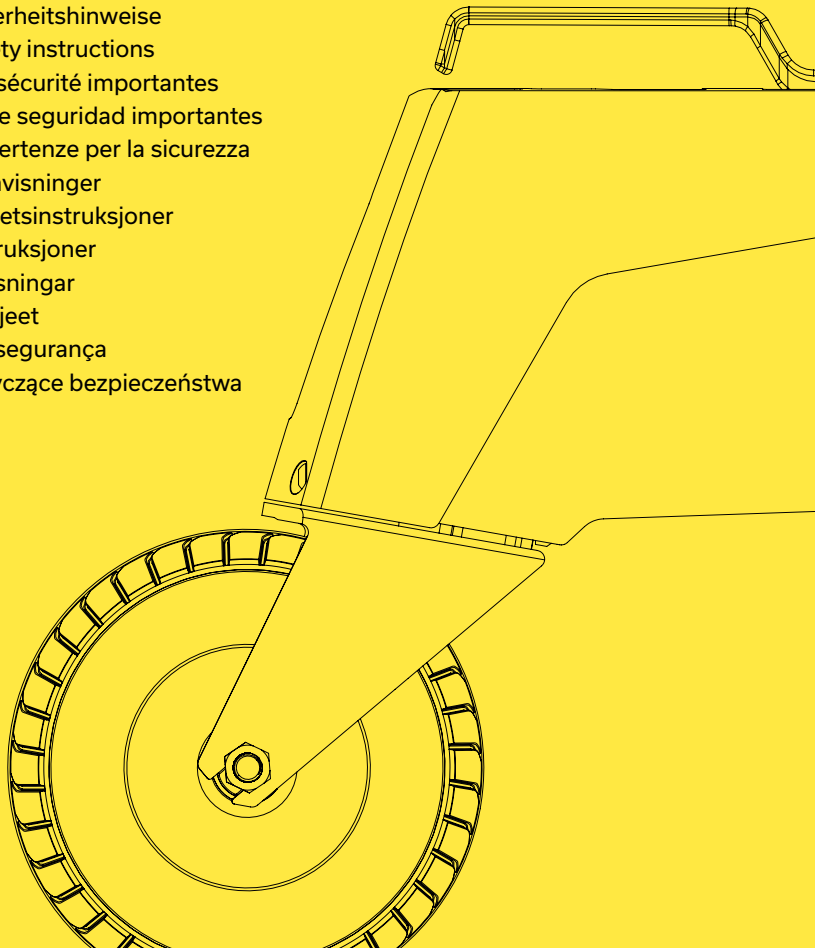

SMOOV one O10

DE	Sicherheitshinweise SMOOV one	4
EN	Safety instructions SMOOV one	6
FR	Consignes de sécurité SMOOV one	8
ES	Indicaciones de seguridad SMOOV one	10
IT	Avvertenze per la sicurezza SMOOV one	12
DK	Sikkerhedshenvisninger SMOOV one	14

NL	Veiligheidsopmerkingen SMOOV one	16
NO	Sikkerhetsinstruksjoner SMOOV one	18
SE	Säkerhetsanvisningar SMOOV one	20
FI	Turvallisuusohjeet SMOOV one	22
PT	Instruções de segurança SMOOV one	24
PL	Instrukcje dotyczące bezpieczeństwa SMOOV one	26

Sicherheitshinweise SMOOV one

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Safety instructions SMOOV one

- Please read the operating instructions before starting the SMOOV one.
- The SMOOV one may only be fitted to and used on wheelchairs that are listed in the Alber data base. To be able to travel with the SMOOV one, pairs of anti-tippers and parking brakes must be fitted.
- The SMOOV one may only be operated by individuals who have been shown how to use it. They must be able to move and coordinate both hands and arms without significant restrictions. They must be in a physical and mental state to be able to operate the wheelchair with the SMOOV one fitted and bring it to a halt.
- The drive unit must be completely switched off prior to installation on the wheelchair, during removal from the wheelchair and during transfer of the driver into or out of the wheelchair. The parking brakes on the wheelchair must be applied.
- When using the SMOOV one, the values specified by the manufacturer of your wheelchair (such as the maximum slope, tyre pressure, maximum speed, etc.), as well as the general operating instructions must be observed.
- When travelling on public roads and footpaths, the applicable national requirements are to be taken into account. Use of the „speed“ function to reach assistance speeds of over 6 km/h is only permitted on private land.
- The SMOOV one must be fully switched off to pass potentially hazardous places. The SMOOV one must be switched off on downward slopes. Never drive across downward slopes. You must not travel on escalators or moving walkways with the SMOOV one.
- Pass obstacles carefully, moving forwards and always at a right angle to the obstacle. If necessary, ask someone for assistance.

- During an emergency stop the drive unit must first be deactivated via the control unit, then the wheelchair can be brought to a standstill via the push rim.
- During intensive use the drive unit may overheat. Do not touch the parts of the drive unit (risk of burns) and allow the drive unit to cool.
- Never place your hands in the swivelling range of the drive wheel (crushing hazard).
- The SMOOV one may only be operated at ambient air temperatures of between -25 °C and +50 °C and may only be charged at between 0 °C and +40 °C. The drive unit and control unit must not be exposed to heat or fire.
- Only use the Alber battery charger provided to charge the battery. Never insert any metal objects into the charger socket on the drive unit. The battery charger must not be used by small children and may only be operated using a mains voltage of 100V–240V. Do not cover the battery charger during charging and do not place any objects on the charger.

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by Alber GmbH