# Aquatec® Sansibar

- EN** Folding shower seat  
User Manual
- DE** Duschklappsitz  
Gebrauchsanweisung
- FR** Siège de douche mural  
Mode d'emploi
- IT** Seggiolino da doccia ribaltabile  
Istruzioni per l'uso
- ES** Asiento plegable de ducha  
Manual de instrucciones
- NL** Opklapbaar douchezitje  
Gebruiksaanwijzing
- DA** Bruseklapsæde  
Brugsanvisning
- FI** Käännettävä suihkutuoli  
Käyttöohje
- NO** Dusjklappsete  
Brukerveiledning
- SV** Fällbar duschsits  
Bruksanvisning
- PT** Assento rebatível para duche  
Manual de instruções

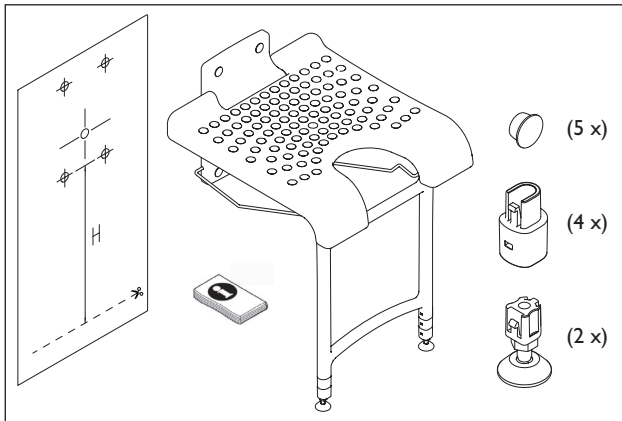
Fig. 1

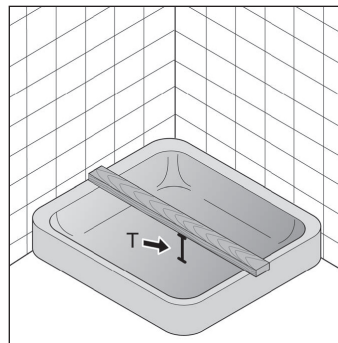
Fig. 2

Fig. 3

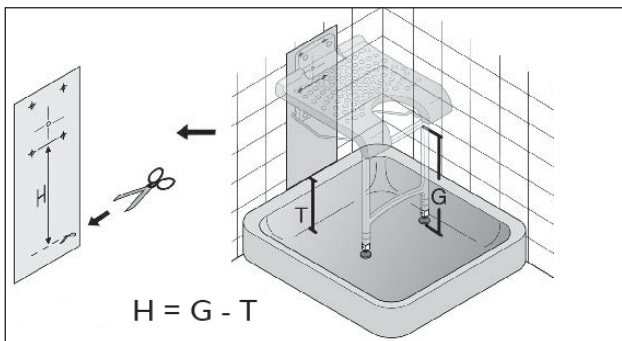
Fig. 4

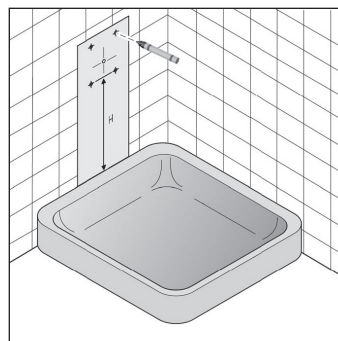
Fig. 5

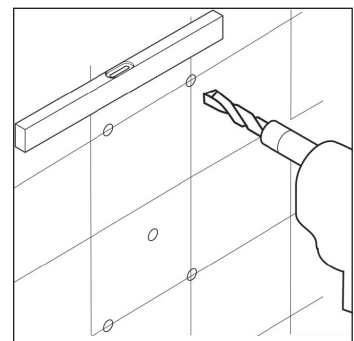
Fig. 6

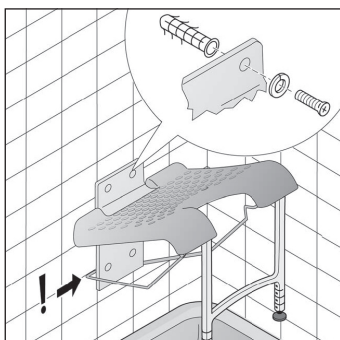
Fig. 7


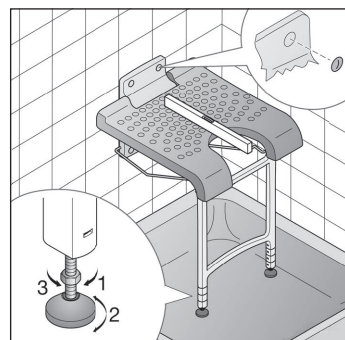
Fig. 8

This manual must be given to the user of the product.  
Before using the product, read this manual and save it  
for future reference!

**AQUATEC**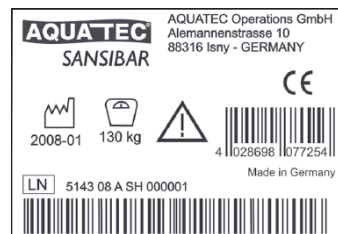

Yes, you can.®

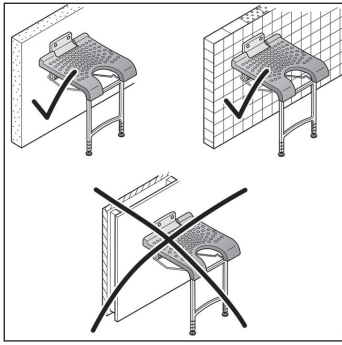
Fig. 9

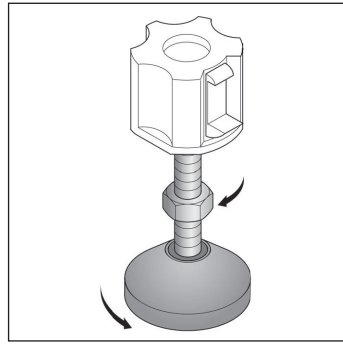
Fig. 10

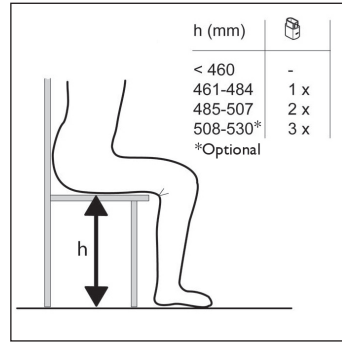
Fig. 11

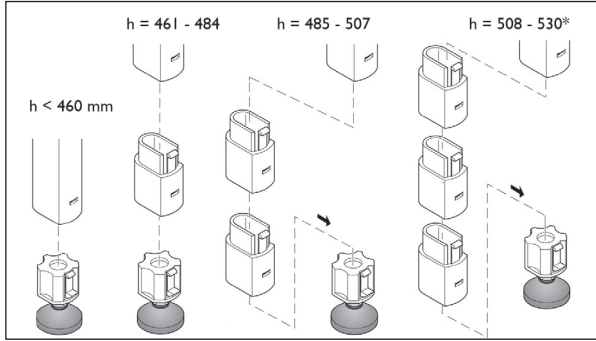
Fig. 12

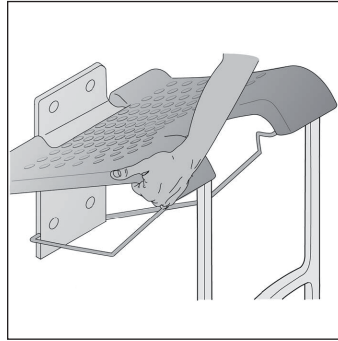
Fig. 13

## Intended Use

This product is designed solely as an aid for taking a shower and for use in the bathroom. Any other use is prohibited.

### WARNING Risk of falling!

- Do not stand on this product.
- Do not use as a climbing aid.

Operating temperature: 10 - 40 °C

## Service Life

The expected service life of this product is three years when used in accordance with safety instructions, maintenance intervals and intended use, stated in this manual. The effective service life can vary according to frequency and intensity of use.

## Safety

### WARNING Risk of injury!

- Do not use if defective.

## Scope of delivery

- Fig. 1.

## Assembly

1. Measure the depth T of the shower tray (Fig. 2).
2. Measure the optimum seat height h of the user (Fig. 11).
3. Use the adapters to adjust the height of the supporting legs to the required seat height h (Fig. 12).

The total height G of the supporting legs of the folding shower seat is 291 mm (+ 23 mm per adapter) (Fig. 3).

4. Deduct the depth T of the shower tray from overall height G.
5. Measure the resulting height H on the template and cut the template horizontally (Fig. 4).
6. Place the template on the wall so that the cut edge is flush with the upper edge of the shower tray (Fig. 5).

### IMPORTANT

- Make sure that the mounting surface on the wall is even.

7. Use a scribing tool to mark the five mounting points indicated by the template (Fig. 5).
8. Use a drill to make suitably sized holes at the points marked on the wall (Fig. 6).
9. Use suitable materials (wall plug, washer and screw) to fix the folding shower seat to the wall (Fig. 7).

### WARNING Risk of overturning!

- Make sure that the pivot mechanism at the back sits correctly into the groove on the mounting plate (Fig. 7).
- The seat should only be mounted on walls of sufficient load-bearing capacity using suitable mounting materials (Fig. 9).

10. Place the covers supplied over the holes (Fig. 8).

11. Using a spirit level, adjust the adjustable feet so that the surface of the seat is exactly level (Fig. 8 / 10).

## Use

### WARNING Risk of overturning!

- Ensure that both supporting legs are set to the same height.

### CAUTION Danger of injury!

- When folding the seat down, ensure that your feet do not get trapped under the legs.

- Fig. 13.

Fold up the folding shower seat after use.

## Maintenance and periodic inspection

The product is maintenance-free, provided the cleaning and safety instructions outlined above are adhered to.

- Check product regularly for damage and to ensure that it is securely assembled.

## Cleaning and disinfection

The product can be cleaned and disinfected using commercially available agents.

- Disinfect the product by wiping down all generally accessible surfaces with disinfectant (after disassembling the product, if possible).

All cleaning agents and disinfectants used must be effective, compatible with one another and must protect the materials they are used to clean. For further information on decontamination in Healthcare Environments, please refer to 'The National Institute for Clinical Excellence' guidelines on Infection Control [www.nice.org.uk/CG002](http://www.nice.org.uk/CG002) and your local infection control policy.

- Clean by hand.
- Clean without scouring agent.
- Max. cleaning temperature 60 °C.

## Re-Use

The product is suitable for repeated use. The number of times it can be used depends on how often and in which way the product is used. Before reuse, clean and disinfect the product thoroughly and check the safety functions, reset if necessary.

## Disposal

The disposal and recycling of used devices and packaging must comply with the applicable legal regulations.

## Warranty

We provide a manufacturer's warranty for the product in accordance with our General Terms and Conditions of Business. Guarantee claims can only be made through the relevant specialist dealer.

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Algol-Trehab Oy
Aquatec Operations GmbH